electric breast pump user guide

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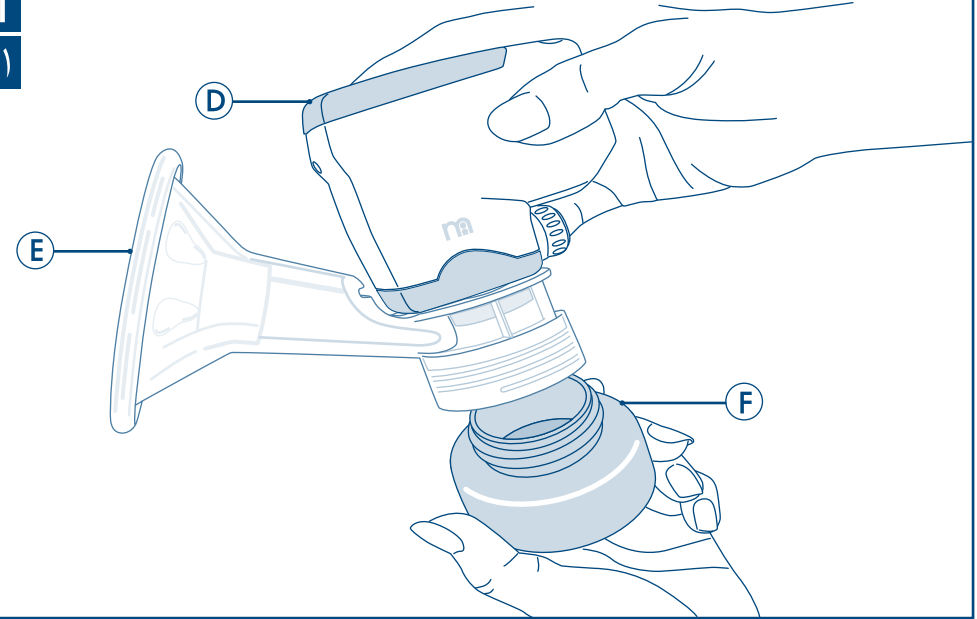
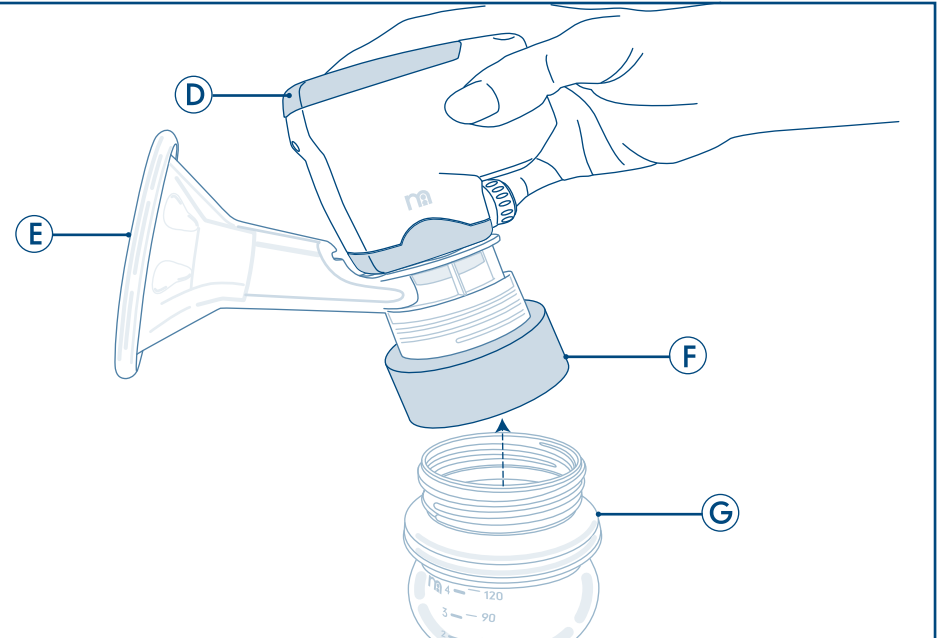
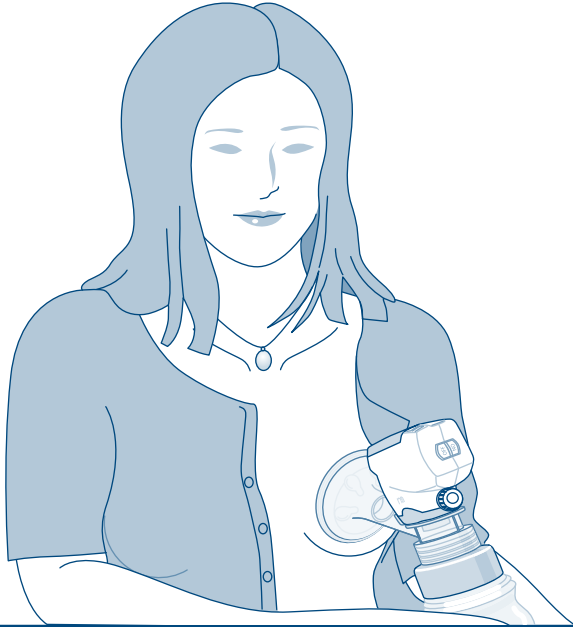
13
١٣

Do not immerse battery operated products. Wipe clean only.

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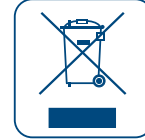
Help the environment by disposing of your products responsibly. The wheelie bin symbols indicate the product and batteries must not be disposed of in the domestic waste as they contain substances which can be damaging to the environment and health. Please use designated collection points or recycling facilities when disposing of the item or batteries.

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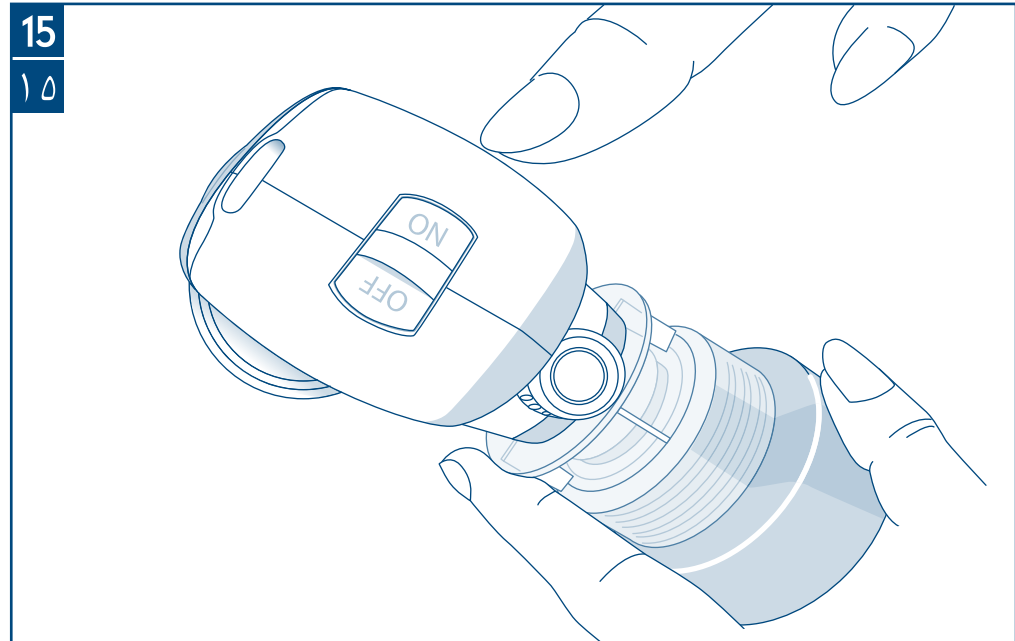
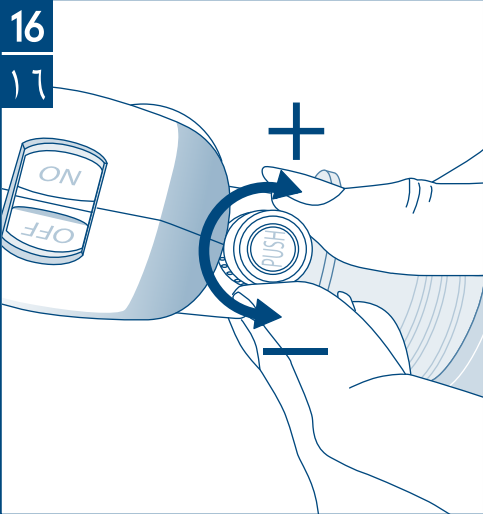
The suction regulator can adjust the suction level when pumping

Turn clockwise to increase the vacuum level, turn anti-clockwise to decrease the vacuum level.

Choose the vacuum level which works best for you and gently press the suction release valve button ("PUSH") for 2 to 3 seconds. Release the button to regain vacuum. This action mimics a baby's natural suck-release-relax cycle and also can massage your breast so as to supply more milk. Repeat this pumping action until required milk is collected. (Massage the breast with the other hand at the same time for pumping result.) When breast milk collection is completed, press the suction release valve button (PUSH) to release vacuum and turn off the power.

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cleaning instructions for pumping unit

Clean the pumping unit after each use according to the following steps.

1. remove the pumping unit (1) from the funnel (3)
2. Fix the connector to the base of the pump
3. Place the connector affixed to the pump in a bowl of warm water, making sure the discharge hole is outside the bowl.
4. When ready turn on the pump for about 30 seconds until you see from the discharge hole clear water coming through.
5. If need be, repeat steps 3, and 4.

safety & care notes

IMPORTANT, RETAIN FOR FUTURE REFERENCE : READ CAREFULLY

Electric Breast Pump

Keep out of children's reach.

Remember to wash and sterilise all parts of your breast pump (and feeding equipment) before use.

Wash the product thoroughly in hot, soapy water or in the top rack of the dishwasher.

Do not allow to come in contact with solvents or harsh chemicals, this could damage the product.

Do not use whilst bathing or showering.

Keep openings free of lint, hair or other foreign materials.

Only use replacement parts and accessories supplied by Mothercare.

Breast milk should be kept at room temperature for the shortest time possible and refrigerated immediately after expression.

Never defrost or heat milk in a microwave.

Frozen milk can be kept for up to three months; once thawed, the milk should be kept refrigerated and consumed within 12 hours.

Once thawed, do not re-freeze milk.

Do not boil breast milk.

Throw away any leftover milk after a feed.

Do not put the motor case or power supply in a microwave.

Always unplug this product immediately after use.

This product should not be left unattended when plugged in.

Do not use outdoors with AC adapter or operate where aerosol spray products are used or where oxygen is being administered.

Do not operate this product if the cord or plug is damaged, if the unit is not working properly, if it has been dropped or damaged or immersed in water -return the product to the manufacturer for examination and/or repair.

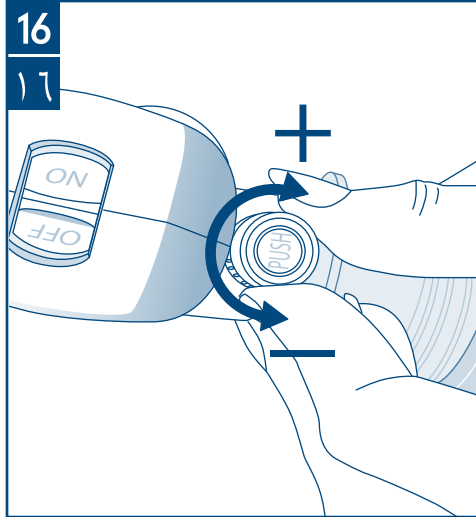
Ensure the electric cord is kept away from heated surfaces.

customer care

If you have a problem with this product please contact your nearest Mothercare store or telephone the Mothercare Customer Care Line on 08453 30 40 30.

Alternatively write to:
Customer care
Mothercare
Cherry Tree Road
Watford
Herts. WD24 6SH
www.mothercare.com

assembly instructions, إرشادات التجميع, instructions de montage, Montageanweisungen, οδηγίες συναρμολόγησης, 组装说明书, instrukcje montażu, инструкции по сборке, instrucciones de ensamblaje, Kurulum talimatları.

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