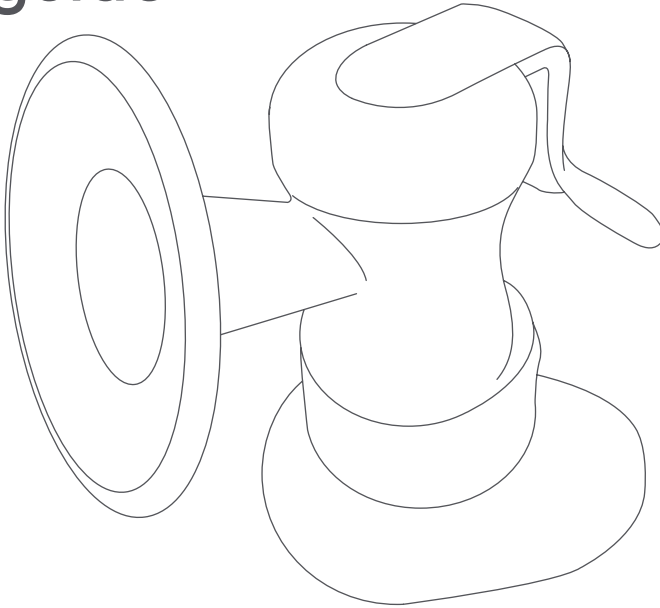
manual
breast pump
user guide

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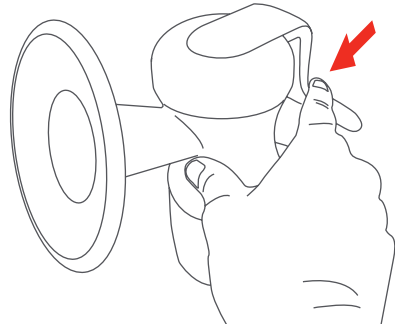
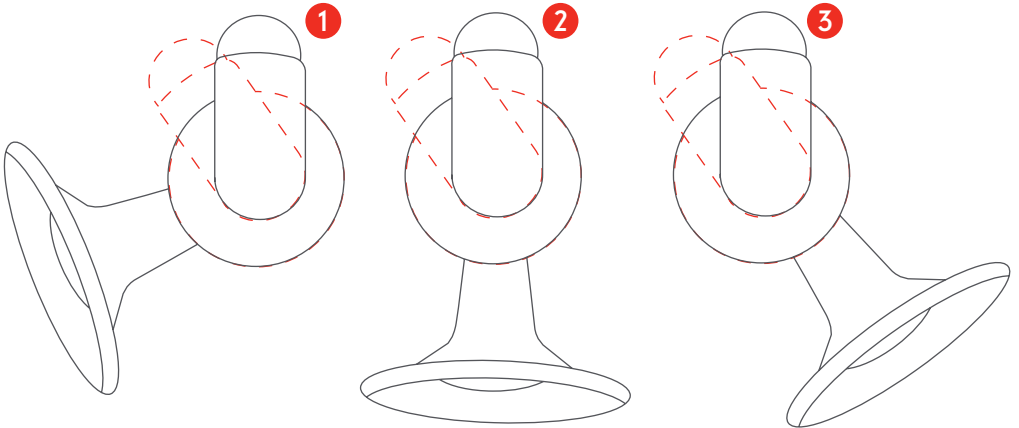
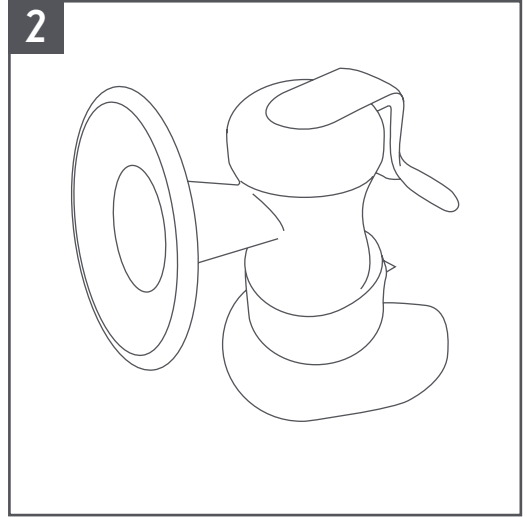
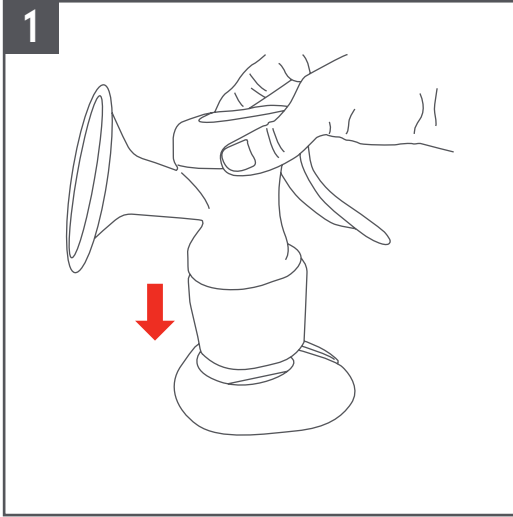
IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY

WARNING! Do not use the pump unit in Microwave.

WARNING! Keep all components not in use out of the reach of children.

WARNING! This is a single user product. Use by more than one person may present a health risk.

WARNING! Never use while pregnant, as pumping can induce labour.

Before first use place in boiling water for 5 minutes, this is to ensure hygiene.

Wash thoroughly before each use and sterilise.

Wash your hands and ensure work surfaces are clean before contact with sterilised components.

Do not store or leave in direct sunlight or near a source of heat, or leave in sterilising solution for longer than recommended, as this could damage the product.

Only use the pump with the Mothercare Innosense feeding bottles.

care of your product

Wash, rinse and sterilise the appropriate pump parts before each use.

Wash the product thoroughly in hot, soapy water or in the top rack of a dishwasher.

Dishwasher safe (max. 65° C) - food colours in the dishwasher may discolour the product.

Do not allow to come into contact with solvents or harsh chemicals, this could damage the product.

Wash your hands and ensure work surfaces are clean before contact with sterilised components.

Do not use bleach, abrasives or antibacterial materials or scourers to clean the product.

Do not store or leave in direct sunlight or near a source of heat, or leave in sterilising solution for longer than recommended, as this could damage the product.

Keep the air openings free of lint, hair, and other foreign material.

customer care

Your child's safety is your responsibility.

If you have a problem with this product or require replacement parts, please contact your nearest Mothercare store or telephone the Mothercare Customer Care Line on 0844 875 5111.

Alternatively write to:

Customer Care
Mothercare
Cherry Tree Road
Watford
Herts. WD24 6SH

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SCREEN COLOURS	R		G		B	
PROCESS COLOURS	C		M		Y	K
SPOT COLOURS	P Cool Grey 11, P485 Red					

IMAGE REFERENCE	Actual DPI	Effective DPI

Artworker	Date	Artworker	Date	Artworker	Date
Steve M	05.12.13				

Artworker Notes

Empty text area for artworker notes.

QC Sign Off	Studio	Production	Spot Check
Artwork size correct			
Bleed added			
Legend all detail checked			
Colour correct to brief/estimate			
Correct logos			
Correct image ref & position			
Colour breakdown correct			
Layers correct			
Point size correct			
Kerning and trapping correct			
Spell checked/grammar using both program and manual check			
Ensure artwork reflects creative templates			
Code positioning and size correct			
Code naming convention on Artwork and legend correct and the same			
File naming convention correct to standard			
Proof read			
Studio Signed:		Date:	
Production Signed:		Date:	
Spot Checked:		Date:	